



## User's instruction for dermatoscope BIO\_BS3

### Package

- 1 x dermatoscope magnifier BS3
- Black Cloth cover
- White box



### How to use

- 1.Install 2pc AA battery into device.



- 2.Press the power button for two seconds to turn on.



- 3.Observe objects which will be displayed under the glass,  
Objects size can be measured by the ruler beside.



- 4.Press the power button again to turn off and put back to the box.

Thank you.

---

地址: 江苏无锡鸿桥路 801-408

电话(Tel): 86-510-85572789

传真(Fax): 86-510-85572789

邮编(Postal code): 214072

Add: Room 408, No. 801 Hongqiao road,

Wuxi, Jiangsu, P. R. China 214072

Email: [sales@cnbiomedical.com](mailto:sales@cnbiomedical.com)